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KIERO Bond – Instructions for use

Product description and use

Kiero Bond is a clear LC liquid to get a bonding between no cured LC composites and acrylics. It creates a inhibition layer in the objects to be bond. It is suitable for the modelation of dental works while using resin attachment. First you apply a fine layer of *Kiero Bond* and then you activate it by means of a standard light curing device between 320 and 550nm. Now you can work the no cured LC composite over the receptive surface and afterwards you light cure it as described in the manual of the manufacturer.

Composition

Composition of acrylic resin and initiators.

Application area

- Bonding between LC composites already polymerized and such which are still not polymerized.
- Bonding between acrylic parts (attachments) or molding discs and not polymerized LC composites

Polymerization

The polymerization of *Kiero Bond* is done by standard light devices between 320 and 500 nm. It polymerizes without vacuum. The polymerization times are:

- 45 sec. with stroboscope light
- 60 sec. with halogen light
- 150 sec. with UVA lamps

Storage instructions

Store between 4 and 25 °C. Keep the bottle closed. Protect the bottle of heat and direct sunlight. Do not use after expiry date.

Delivery unit

5ml in a light impervious bottle

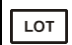
This product was developed for dentistry use and should be used according to the instructions for use. The manufacturer does not accept liability for damage caused by its use for any other purpose. The user is also personally responsible for checking before use that the material is suitable and can be used for the intended purpose, particularly if this is not listed in the instructions for use.

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