**Soporte de mesa de 45°**

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**45° Table Support**

The 45° degree table support if needed can be inserted into the Articulator base

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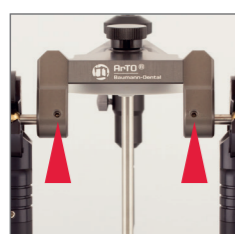
**Centric Adjustment**

If you notice a lateral movement of the condylar axis while the centric locks are closed, the centric position must be adjusted as below

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Notice: If you had calibrated the articulator it is necessary to recalibrate after repairs.

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Loosen the screws by using a 2 mm Allen key.

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Adjust the top to center. Pull out the semi-axis till stop and fasten the screws finger tight.

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To mount a magnetic plate system, please use the enclosed special screw. Fasten the screw finger-tight by using a 5 mm Allen key. If you need to calibrate the articulator, use the Adesso Split calibration key to glue in the bottom plate. Fixing the base plate only by Adesso Split synchronize adhesive #11000. den Baumann Dental Justierkleber #11000.

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**Warning:** It is absolutely essential to check calibrated articulators by using a calibration key. Recalibrate the articulator in order to correct discrepancies by using the calibration key.

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**Cleaning and Care suggestions:** Condylar ball and axis path cleanliness maintains reliability and accuracy. Steam clean followed by pressured air is recommended. For more efficient cleaning use the Baumann Dental Cleaning Spray #19088. Spray over area and allow to soak 1-2 minutes prior steam cleaning. Allow to dry completely. The Baumann Articulator Silicon- Carespray # 11009 is highly recommended to care and protect your Articulator for further longterm usage.

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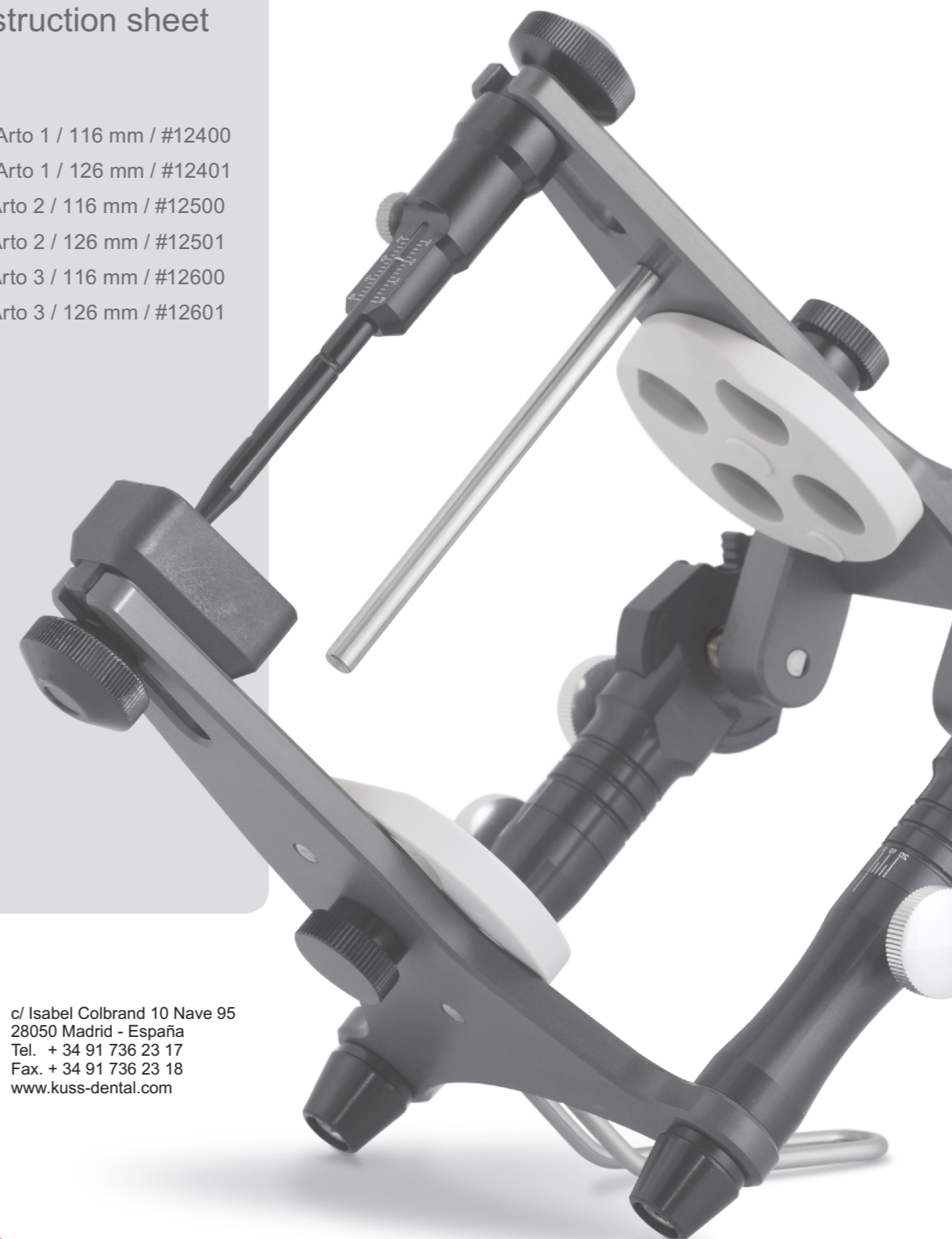
**Note on Safety:** Your Articulator must never come into contact with acid or any solvents, as this would damage the surfaces and parts. Your Articulator must never be ultrasonically cleaned, this would disable its functional use. In case of neglect or misuse your guarantee rights with Baumann Dental GmbH will void

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




**KUSS DENTAL**  
Innovative Products  
Made in Europe

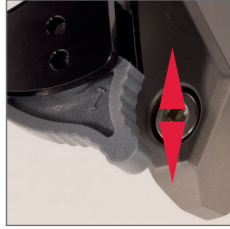c/ Isabel Colbrand 10 Nave 95  
28050 Madrid - España  
Tel. + 34 91 736 23 17  
Fax. + 34 91 736 23 18  
www.kuss-dental.com

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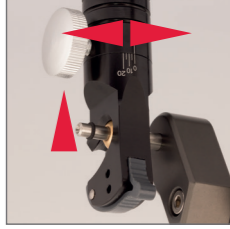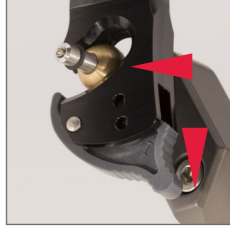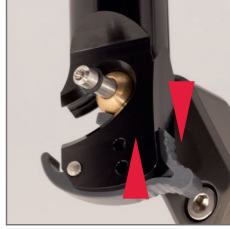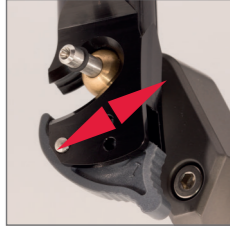**Centric Locking**  
Upwards and downwards Locking / Unlocking mechanism.

The upper part of the articulator once unlocked allows a free condylar movement. The unlocked mechanism keeps the upper part intact.

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To remove the upper part unlock the condylar ball within it's path then lock the mechanism rotation movement only may be carried out.

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**Centric Locking**  
Upwards and Downwards Locking / Unlocking mechanism.

The upper part of the articulator once unlocked allows a free condylar movement. The unlocked mechanism keeps the upper part intact.

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
**Bennett Angle**  
By loosening the screw the Bennett angle can be variably adjusted. Tighten screw after angle adjustment. The Bennett angle may be adjusted from 0° to 20°.

**Condylar Movement**  
The Arto 2 Articulator provides a fixed Condylar Inclination of 35°.

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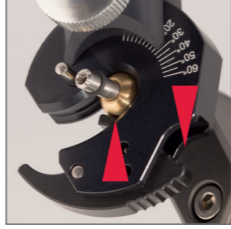**Arbitrary Axis**  
Facebow readings may be transferred directly onto the articulator Condylars axis pins. Please adjust the Bennett angle to 0° and the Condylar Inclination angle to 60°.

### Arto 3

### Arto 3

### Arto 2

### Arto 2

### Arto 1

### Arto 1